## DE ALLGEMEIN:

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## GENERAL USE

GB

**earpad® universel** gives your ears the protection required to help safeguard your hearing from excessive levels of sound that may lead to tinnitus, deafness and other noise overdose related symptoms. The unique **earpad®** acoustic filter will reduce the sound reaching your eardrum yet still allowing you to hear the music with it's full acoustic content enhancing the lows and highs with it's built in passive loudness filter; regard it as a volume control for your ears.

## FITTING

Before fitting your **earpad® universel**, make sure your hands are clean.

Hold the end of the earplug and insert it into your ear canal until you feel it fit snugly into the contours of your ears.

Test the fit. In a noisy environment, and with ear plugs inserted, cup both hands over your ears and release. You should notice a significant difference in the noise level. If the noise seems to lessen when your hands are cupped over your ears, your earplugs are probably not fitted properly. Remove them and refit following these same instructions; Then, you feel the perfect position when **earpad® universel** is well fitted.

## MAINTENANCE

Regularly clean your **Earpad® universel**. After using, place it in the case provided for this purpose. Use wet tissue to clean your **earpad® universel**.

Failing to comply with the instruction for use above may seriously shorten the usefull life and reduce the effectiveness of the earplugs. **EARSONICS** SAS can not be held responsible for these consequences.

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SAS au capital de 100000€ - RCS Montpellier 50111569500014